
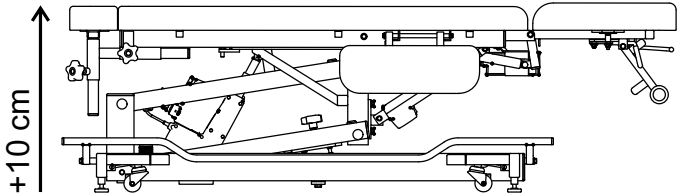
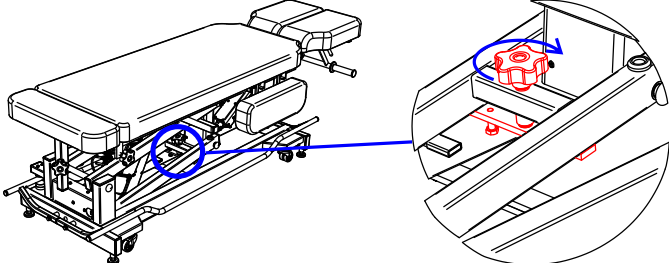
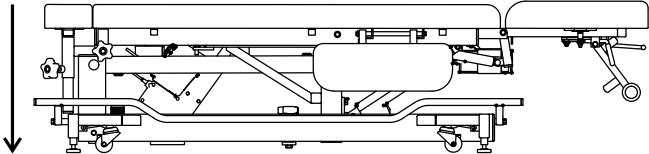
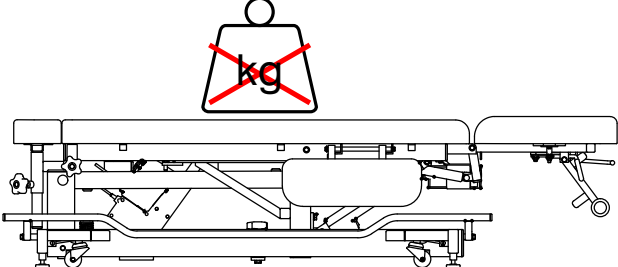
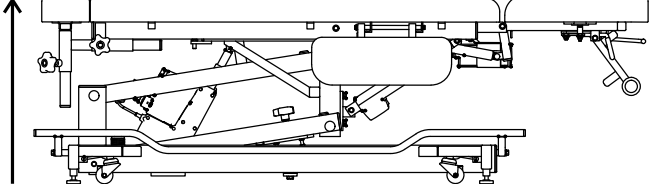


MMM Mécanique Médicale Sàrl

Dispositif de mise sur roulettes

<p>1. La table doit être surélevée de au moins 10 centimètres. <i>Photo 1</i></p>	<p>Photo 1</p>  <p>+10 cm</p> <p>The diagram shows a side view of the table mechanism. An upward-pointing arrow on the left is labeled '+10 cm', indicating the required height of the table above the wheels.</p>
<p>2. Positionner la commande. <i>Photo 2</i></p>	<p>Photo 2</p>  <p>The diagram shows a side view of the table with a blue circle around the control lever. A blue line connects this circle to a magnified inset on the right, which shows the lever being moved from its rest position to the 'on wheels' position, indicated by a red arrow.</p>
<p>3. Baisser complètement la table. <i>Photo 3</i></p>	<p>Photo 3</p>  <p>The diagram shows a side view of the table mechanism. A downward-pointing arrow on the left indicates the table is being lowered to its lowest position.</p>
<p>4. La table peut à présent se déplacer, sans autre charge que son propre poids. <i>Photo 4</i></p>	<p>Photo 4</p>  <p>The diagram shows a side view of the table. Above the table is a warning symbol: a trapezoid with a circle on top and the letters 'kg' inside, crossed out with a red 'X'.</p>
<p>5. Après son déplacement relever la table. <i>Photo 5</i></p>	<p>Photo 5</p>  <p>The diagram shows a side view of the table mechanism. An upward-pointing arrow on the left indicates the table is being raised.</p>
<p>6. Positionner la commande de mise sur roulette. <i>Photo 6</i></p>	<p>Photo 6</p>  <p>The diagram shows a side view of the table with a blue circle around the control lever. A blue line connects this circle to a magnified inset on the right, which shows the lever being moved from its rest position to the 'on wheels' position, indicated by a red arrow.</p>

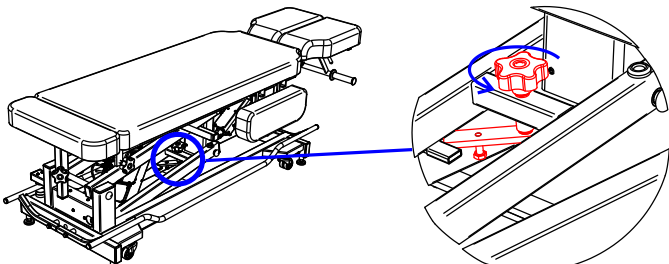

ATTENTION RISQUE DE BLESSURE DU PATIENT!

La table ne doit pas être sur ses roulettes lors du positionnement du patient.